

***In Vitro* protocol in brief**

CustomPanel/ThemaPanel tests

1. Reverse transcription

Reagents:

- ☐ Kit M-MLV RT
- ☐ dNTPs (10mM)
- ☐ Hexamers (100μM)
- ☐ ARN to be tested

Steps :

- ☐ Thaw the reagents
- ☐ Prepare reverse transcription mix
 - RT buffer (1,25 μL)
 - DTT (0,5 μL)
 - dNTPs (1 μL)
 - Hexamers (1 μL)
- ☐ Deliver 3,75 μL mix per tube
- ☐ Add 2 μL of RNA sample
- ☐ Vortex and centrifuge
- ☐ Put samples in thermocycler and start program 1 step 1a
- ☐ Add 0,5 μL of M-MLV RT
- ☐ Centrifuge
- ☐ Put samples in thermocycler and start program 1 step 1b
- ☐ Once the program is over and the block cool down at 4°C, put out tubes
- ☐ Put tubes on ice or cooling block



Then proceed to step 2 or store cDNA products between -30°C et -15°C.

2. Hybridization of probes

Reagents:

- ☐ SALSA MLPA Buffer
- ☐ **GEP- CPPM** probe Mix

- ☐ Thaw reagents
- ☐ Prepare Hybridization mix
 - Salsa MLPA Buffer (1,5 μL)
 - GEP-CPPM** Probe Mix (1,5 μL)
- ☐ Vortex and centrifuge
- ☐ Deliver 3 μL of mix per cDNA tubes
- ☐ Centrifuge
- ☐ Put samples in thermocycler and start program 1 step 2

3. Ligation

Reagents:

- | | |
|--|---|
| <ul style="list-style-type: none"> <input type="checkbox"/> SALSA Ligase Buffer A <input type="checkbox"/> SALSA Ligase Buffer B <input type="checkbox"/> SALSA Ligase 65 <input type="checkbox"/> Nuclease free water | <ul style="list-style-type: none"> <input type="checkbox"/> Thaw reagents <input type="checkbox"/> Prepare ligation mix <ul style="list-style-type: none"> <input type="checkbox"/> Nuclease free water (25 µL) <input type="checkbox"/> Salsa Ligase Buffer A (3 µL) <input type="checkbox"/> Salsa Ligase Buffer B (3 µL) <input type="checkbox"/> Vortex and centrifuge <ul style="list-style-type: none"> <input type="checkbox"/> Salsa Ligase 65 (1 µL) <input type="checkbox"/> After 1h of hybridization, deliver 32 µL of mix per cDNA tubes <input type="checkbox"/> Start program 1 étape 3 |
|--|---|



Then proceed directly to step 4 or store the ligation products between -30°C and -15°C.



After this step, do not store the products at high temperature (e.g. 4°C or at room temperature) in order to avoid non-specific ligations which could result from a residual activity of the enzyme.

4. Amplification and incorporation of barcodes and adaptators

Reagents:

- | | |
|---|---|
| <ul style="list-style-type: none"> <input type="checkbox"/> GEP-BC-xxx Barcodes <input type="checkbox"/> Q5 MasterMix <input type="checkbox"/> Nuclease free water | <ul style="list-style-type: none"> <input type="checkbox"/> Thaw the reagents <input type="checkbox"/> Prepare PCR mix <ul style="list-style-type: none"> <input type="checkbox"/> Q5 MasterMix (12,5 µL) <input type="checkbox"/> Nuclease free water (5,5 µL) <input type="checkbox"/> Vortex and centrifuge <input type="checkbox"/> Deliver 18 µL of mix per wells of PCR plate <input type="checkbox"/> Add 5 µL of ligation product in each well <input type="checkbox"/> Add 2 µL of barcode (GEP-BC-xxx) in each well |
|---|---|



Use different barcodes BEP-BC-xxx for each tested samples.

- ☐ Put samples in thermocycler and start program 2



Then proceed directly to step 5 or store the PCR products between -30°C and -15°C.



Do not store these products for prolonged periods at high temperature (e.g. 4°C in the thermocycler or at room temperature).

5. Purification and dosage sequencing librairies

Reagents:

- ☐ 100% Ethanol
- ☐ Nuclease free water
- ☐ AMPure XP Magnetic beads
- ☐ TE Buffer
- ☐ Qubit dsDNA HS Assay

Steps :



Ensure beads are completely re-suspended before use.

- ☐ Purify 25 μ L products with 45 μ L AMPure XP
- ☐ Elute PCR products in 50 μ L of TE buffer



After purification, librairies can be stored between -30°C et -15°C before sequencing.

- ☐ Dose 10 μ L of each library by fluorimetry



This rapid protocol completes the notice. It does not dispense with the complete reading of the notice.